


Instructions for Use


Fentanyl Test Strips

For External Use Only / Test for Fentanyl Before Consumption

Each Pouch Contains:
1 Test Strip &
1 Microscoop

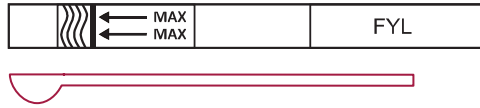


You Will Need:
Sample
Bottle Cap
Water (1 tsp)

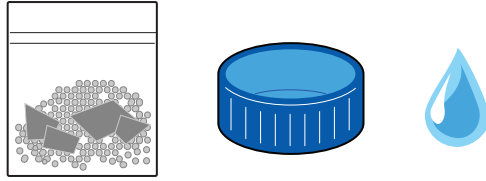


Step 1: Sample Preparation

1 Crush & Fill

Crush and mix sample thoroughly by shaking sample.

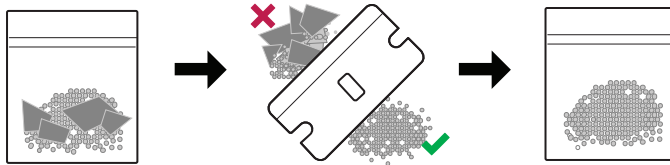


Fill 1 bottle-cap with 5ml of water

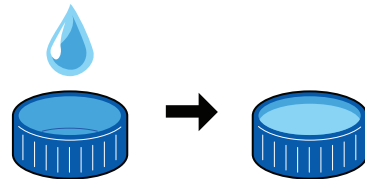

2 Add & Stir

Collect 3 scoops of sample.


Fill bottlecap with 10mg powder and stir for 15 seconds.

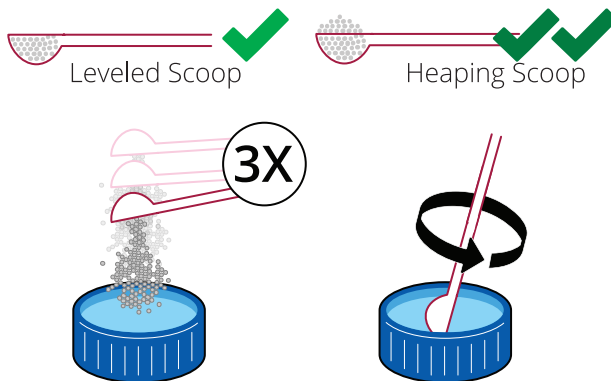

3 Dip & Read

Dip test strip into the bottle cap to the max line for 15 secs.

Read results after 1 min.

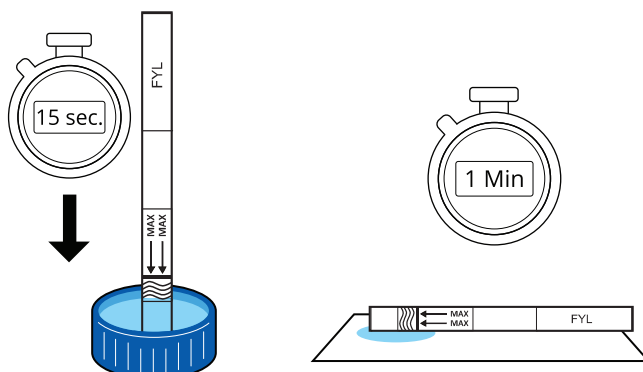

Bigger sample, Better Detection

Best Method: Test Everything.

Due to the dangerous 'chocolate chip cookie' effect, where fentanyl is unevenly distributed in powders and pills. We strongly urge you to test everything before use to protect your safety.

Backup Method: Test a majority of the drug before consumption.

Different Sample Prep Methods

Crystal/powdered drugs

Measure your sample and use the 10 mg microscoop or a milligram scale. Place sample in a small container. For 10-50 mg (1 - 5 microscoops): a bottle cap

Pressed pills/tablets

Crush the pill(s) into fine powder and refer to Step 1

For other formats, blotter LSD or IV Drugs, please visit:
www.tddsolutions.org/FTS


Interpreting The Results



No Fentanyl



Invalid

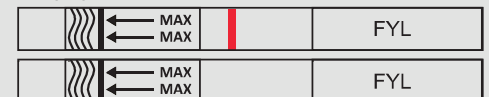

Fentanyl: ONLY 1 colored line appears in control region.

NO Fentanyl: 2 colored lines appear. 1 in control region(C) & 1 in test region (T).

*Even faint lines are considered.

Invalid: As control line fails to appear.

For more detailed instructions & information please visit:
www.tddsolutions.org/FTS

Manufactured for:

